

Standard Work | Intermountain Healthcare

Performed By: Imaging Staff

Key Process:

ISOLATION Cleaning:

Imaging Portable Procedures

<u>Trigger</u>: Isolation Patient Exam <u>Owner</u>: EVS/Img <u>V</u>: 1.1 3/23/20

Next Review April 2021



Timing	Major Steps	Key Points	Notes
At end of Procedure, While in Patient Room	 Dispose of cassette covers and other used supplies in patient room 	Dispose of waste as you normally woud via regular trash, red bag waste, sharps etc.	
While Exiting the isolation room	 Doff PPE just inside the patient room Don Clean gloves. Exit the room with the portable equipment 	 Move the portable equipment to the edge of the room near the door. Follow Donning/Doffing guidelines in PPE/Masking Toolkit on Intermountain.net 	
AFTER Exiting the isolation room	5. Don appropriate clean PPE	Gloves always Gown and Eye protection if splash is possible or if gross contamination exists	
	6. BEGIN CLEANING AND DISINFECTION PROCEDURE	1. If any surfaces are soiled, you must clean the surface first and then disinfect with the appropriate disinfectant wipes.	Clean the equipment as near the isolation area as possible to avoid potential contamination by traveling through hallways, in elevators, etc.
	7. Use appropriate disinfectant wipes	1. Disinfectant must stay wet and be allowed to sit for the specified wet dwell time to be effective. Resaturate if disinfectant evaporates too soon.	Purple top = 2 min Gray top = 3 min Bleach = 4 min (required for C-diff)
	8. Surfaces, controls, monitors, accessory equipment, etc.	1. Maintain Wet time	Always clean from top to bottom and cleanest to dirtiest
	9. Wheel treads	Only need to be cleaned if rolled through a large body fluid spill.	Only cleaning (not disinfection) is necessary.
Visual Cu	es: \star Critical Steps	Safety Timing Tip	