



Infusion Pump Cleaning Log

Infusion pumps used in Medical Group areas are “W” wiped down between each patient.

A “D” deep clean is needed once a day (or once in a 24-hour period) if the infusion pump is utilized.

- Wiping down the pumps for high-touch cleaning is done with the **purple top PDI wipes (2 min dry time)** and does not include cleaning the IUI connectors.
- **Bleach wipes (4 min dry time)** are used for patients with C-diff, Norovirus, and Candida auris. After the appropriate 4 min dry time, the pump is wiped down with a damp cloth to remove residue.
- **Grey top wipes (3 min dry time)** are not appropriate for pump cleaning use.
- Ensure dry times are completed before connecting modules to the Point of Care Unit (PCU).

Dept/Facility: _____ Month/Year: _____

Infusion Pump Serial or Identifying number _____

**** If clinic has multiple pumps in use, please print multiple logs and label appropriately.**

Date/time	Circle W for “Wipe Down” Circle D for “Deep Clean” and continue with checkmarks in the following columns to show Deep Clean components were met	Attach IUI Connector Cover	Equipment should be upright, unplugged, and turned off before cleaning	Use a soft bristle brush to clean the case removing any visible residue	Use Purple or Orange top wipes as appropriate to clean device surfaces. Then wipe the with a clean damp cloth to remove residue from using sani-wipes on the equipment.	Remove IUI Connector cover. Use 70% isopropyl alcohol directly to the dedicated IUI cleaning soft bristle brush (DO NOT USE THIS BRUSH FOR ANYTHING ELSE) To prevent contamination, do not dip the brush into the alcohol container	Clean IUI’s with the dedicated IUI cleaning brush UP and DOWN. Do not brush them side to side as this can damage the pins	Inspect surfaces and moving parts for signs of damage such as cracks or broken parts. Open pump Module and PCA Module doors and inspect the interior parts.	Inspect the bottom of each PCU to check if any of the 4 battery screws or washers are missing	Let the IUI connectors air dry for 15 minutes before connecting the device to another device.	Initials
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										

Date/time	Circle W for "Wipe Down" Circle D for "Deep Clean" and continue with checkmarks in the following columns to show Deep Clean components were met	Attach IUI Connector Cover	Equipment should be upright, unplugged, and turned off before cleaning	Use a soft bristle brush to clean the case removing any visible residue	Use Purple or Orange top wipes as appropriate to clean device surfaces. Then wipe the with a clean damp cloth to remove residue from using sani-wipes on the equipment.	Remove IUI Connector cover. Use 70% isopropyl alcohol directly to the dedicated IUI cleaning soft bristle brush. To prevent contamination, do not dip the brush into the alcohol container	Clean IUI's with the dedicated IUI cleaning brush UP and DOWN. Do not brush them side to side as this can damage the pins.	Inspect surfaces and moving parts for signs of damage such as cracks or broken parts. Open pump Module and PCA Module doors and inspect the interior parts.	Inspect the bottom of each PCU to check if any of the 4 battery screws or washers are missing	Let the IUI connectors air dry for 15 minutes before connecting the device to another device.	Initials
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										
	W D										

Caregiver initials are required to verify completion of log. All completing cleaning log should print name and initials below to match initials with caregiver involved.

Name_____ Initials_____	Name_____ Initials_____
Name_____ Initials_____	Name_____ Initials_____
Name_____ Initials_____	Name_____ Initials_____